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VISAID 0.3% MONODOSIS is a lubricating, hypotonic and preservative-free ophthalmic solution with a combination of hydrating polymers (sodium hyaluronate, povidone and glycerine) and essential electrolytes, for ocular administration.

COMPOSITION:

Sodium hyaluronate 0.30 g, Glycerine, Povidone, Sodium chloride, Magnesium chloride, Potassium chloride, Calcium chloride, Boric acid, Borax decahydrate, Purified water 100 ml.

NO PRESERVATIVES

INDICATIONS:

VISAID 0.3% MONODOSIS is intended for:

Long-lasting relief from the symptoms of moderate to severe ocular dryness, and other types of irritations not linked with clinical pathologies, such as burning, itching, eye fatigue or the sensation of a foreign body in the eye. These irritations are caused by various factors such as: dust, smoke, dry environments, air conditioning, prolonged use of computer screens, hormonal imbalance (menopause) or refractive surgery (LASIK), amongst others.

INSTRUCTIONS FOR USE:

Before using the product, wash your hands with soap and water.

Take a vial of VISAID 0.30% MONODOSIS and open it by turning the cap.

Apply one or two drops into each eye and blink several times.

VISAID 0.3% MONODOSIS can be used up to 8

times a day.

Date of last revision: 2015/10

PRECAUTIONS:

Once the vial is open, use immediately. This product does not contain preservatives and must therefore be discarded after use to prevent microbial contamination.

Each vial is for one use only.

Do not use if the blister is damaged.

Store at room temperature.

Do not ingest; keep out of reach of children.

Do not use after the expiry date.

INTERACTIONS:

Do not use in conjunction with any medication or other product that is applied to the surface of the eye, to prevent any alteration of the effects of both products.

CONTRAINDICATIONS:

Do not use the product if you are allergic to any of its components.

ADVERSE EFFECTS:

In case of ocular irritation or adverse reaction please consult a specialist.

AVIZOR S.A
La Cañada 13-17
28850 Torrejón de Ardoz, Madrid (SPAIN)

300183